

## **DAILY QA**

Every 24 hrs = SYSTEM>END>RESTART (Approx. 3-4 min)

Place Phantom in holder. Center crosshairs by touching the head on touch screen on the scanner, then hold blinking blue arrow to center phantom. Verify that crosshairs are centered.

FYI-If you hold finger on the head, it will reset the table to current position.

On console:

Setup>Quality (Follow Instructions)

---

## **WARM-UP**

If scanner hasn't been used in 2 HRS-

Calibration or Check-up, but calibration is shorter

SETUP>CALIBRATION

---

## **WEEKLY QA**

Sunday Clean up= SYSTEM>RUN>CLEAN UP AND SHUT DOWN

Once the scanner is completely shut down, GREEN LIGHT on the control panel (top right) will be solid (not blinking).

POWER ON- top left control panel, 3rd button from top

---

## **Additional Info**

HARD RESTART- top left control panel, 2nd button from top (do not used unless told by service)

*Try CTRL-ALT-DELETE first in case of issues*